

ConductiveBackBrace MEDICAL Better Living™

QUICK REFERENCE GUIDE

PAGE 1



The purpose of this guide is to assist you in correctly operating and caring for your conductive back brace garment. This device should only be used under the continued supervision of a licensed healthcare professional. **PLEASE READ THIS INSTRUCTION MANUAL COMPLETELY BEFORE USING THIS GARMENT.** Read, understand and practice the warnings and operating instructions. Always follow the operating instructions prescribed by your healthcare provider.



WARNINGS:

- For single patient use only.
- Do NOT apply to broken skin. Should a skin rash occur, discontinue use and contact your physician.
- Do NOT stimulate while driving a motorized vehicle or operating machinery.
- Conductive garments should be used with electrical sources designed and approved for medical electrical stimulation.
- Do NOT use on patients with pacemakers.
- Do NOT place or operate the garment anywhere besides its area of intended use.
- Remove all jewelry prior to use.
- Always use a conductive spray or water with your garment, or you may experience an unpleasant stinging sensation.


CARING AND CLEANING

- After use, hand wash in cold water only.
- Dry flat at room temperature only. Do not place in a dryer or use an iron on the garment.
- Do not use bleach.




1 APPLY CONDUCTIVE SPRAY OR WATER ON ELECTRODES INSIDE OF GARMENT

Generously moisten the electrodes on the back of the garment with a conductive spray* or with water. The moisture acts as a coupling substance and ensures the effectiveness of the treatment.

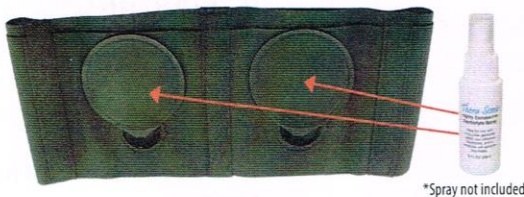

2 PLUG LEAD WIRES INTO RUBBER HOLES OF ELECTRODES

Insert the pins of the lead wires into the holes of both electrodes sewn into the garment. Make sure no metal is visible.

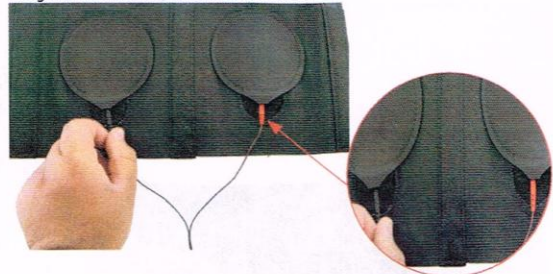


3 PLACE CONDUCTIVE GARMENT ON BACK

Remove all jewelry from the area that will be covered by the garment. Clean the skin thoroughly with soap and water.

Place the Conductive Back Garment on the back with the lead wires facing downward as shown below. Secure into place using the Velcro™ fastener that is sewn into the ends of the garment.

Make sure the garment is placed and conforms directly on the skin.

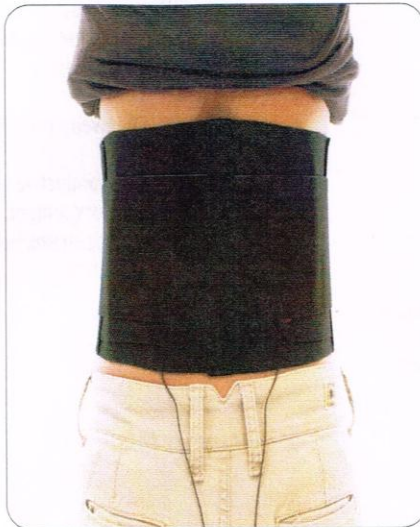

4 PLUG LEAD WIRES INTO STIMULATOR AND BEGIN TREATMENT

Be sure the electrical stimulator is off. Securely plug the lead wires into the stimulator. Following the treatment provided by your practitioner, turn the stimulator on and adjust as directed by practitioner.



5 CLEANING INSTRUCTIONS

After use, hand wash in cold water only. Dry flat at room temperature only. Do not place in a dryer or use an iron on the garment. Do not use bleach.

